
**R-080124**

<b>E.....</b>	<b>CATALIZADOR DE HUMOS PARA HORNOS .....</b>	<b>4</b>
<b>GB.....</b>	<b>CATALYTIC CONVERTER FOR FURNACES .....</b>	<b>8</b>
<b>F.....</b>	<b>CATALYSEUR DE FUMEE POUR FOURS .....</b>	<b>12</b>
<b>P.....</b>	<b>CATALISADOR DE FUMOS PARA FORNOS.....</b>	<b>16</b>
<b>I.....</b>	<b>CATALIZZATORE FUMI PER FORNI .....</b>	<b>20</b>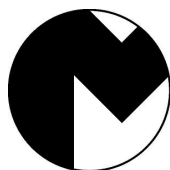

**MESTRA®**

**TALLERES MESTRAITUA S.L.**

Txori-Erri Etorbidea, 60

Tfno. + 34 944530388 - Fax + 34 944711725


E-mail: [mestra@mestra.es](mailto:mestra@mestra.es) - [www.mestra.es](http://www.mestra.es)

**48150 SONDIKA - BILBAO - ESPAÑA**

Rev. 13/12/12

# MESTRA®

CATALIZADOR DE HUMOS PARA HOI  
CATALYSEUR DE FUMÉES POUR FC  
SMOKE CATALYST FOR FURNACE  
R-080124

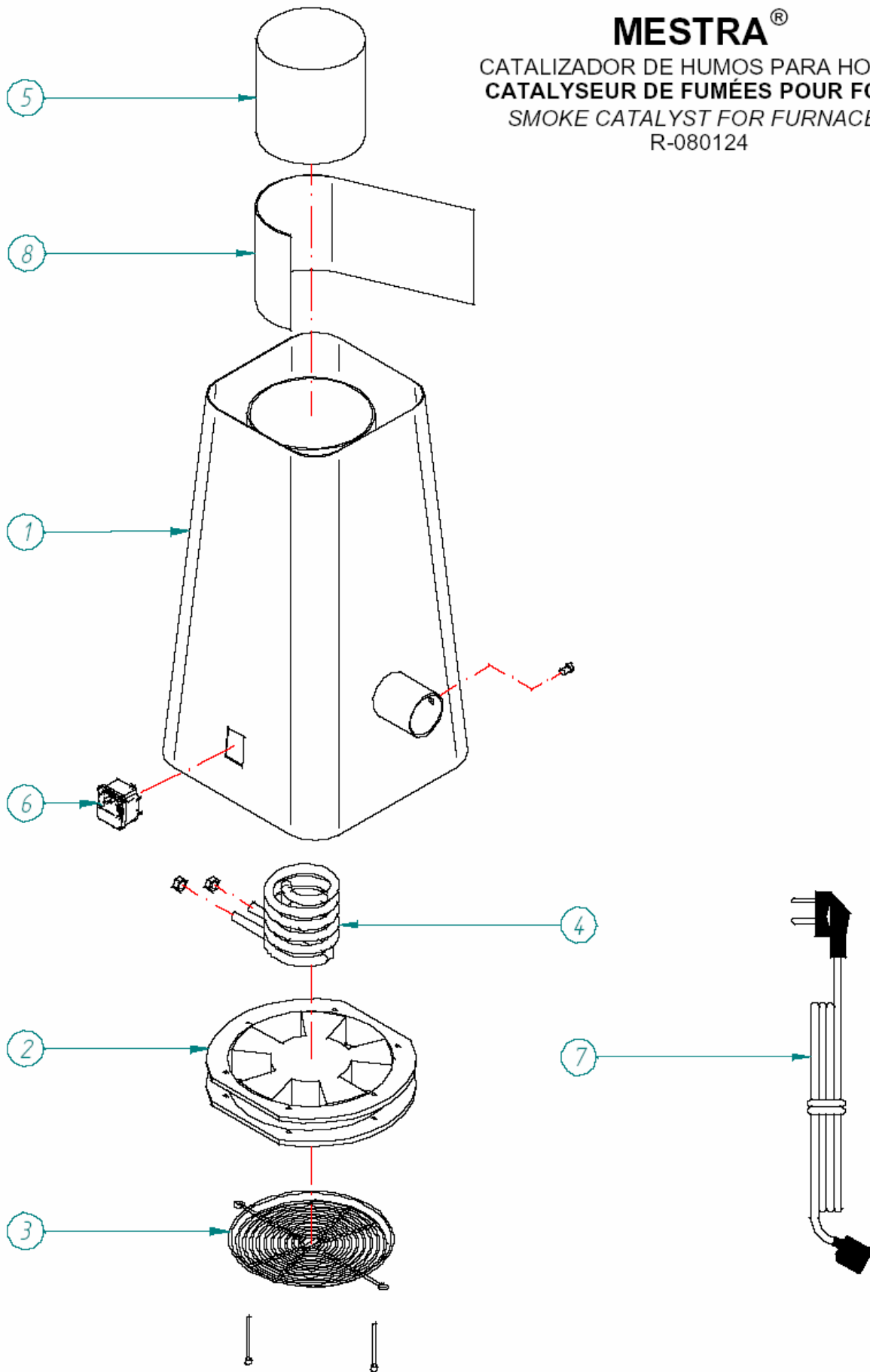1	080124-01	Carcasa / <b>Cover</b> / <i>Carcasse</i> / Carcaça / <b>Telaio</b>
2	080124-02	Ventilador / <b>Fan</b> / <i>Ventilateur</i> / Ventilador / <b>Ventilatore</b>
3	080124-03	Rejilla ventilador / <b>Fan grille</b> / <i>Grille ventilateur</i> / Grelha ventilador / <b>Griglia ventilatore</b>
4	080124-04	Resistencia / <b>Resistor</b> / <i>Résistance</i> / Resistência / <b>Resistenza</b>
5	080124-05	Cartucho catalizador / <b>Cartridge</b> / <i>Cartouche catalyseur</i> / Cartucho catalisador com isolamento / <b>Cartuccia catalizzatore</b>
6	080490-06	Base de conexión con portafusibles / <b>Connection plug with fuse-box</b> / <i>Base de connexion avec porte fusibles</i> / Base de ligação com portafusíveis / <b>Base di collegamento con portafusibili</b>
7	000200	Cable / <b>Power cord</b> / <i>Câble</i> / Fio / <b>Cavo</b>
8	080124-08	Fibra aislante / <b>Insulator</b> / <i>Isolant</i> / Isolamento / <b>Isolamento</b>

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## INTRODUCTION

The R-080124 Catalytic Converter is a simple device that can be fitted to the exhaust hose of the pre-heating furnace in order to remove the gases and non-burnt products generated as a consequence of the combustion of waxes and other materials.

The ventilation system removes the gases produced inside the furnace and takes them to the extractor hood placed outside. Inside the converter, the gases and non-burnt materials are reheated again and reduced to CO<sub>2</sub> and H<sub>2</sub>O. This system gets rid of most of the bad odours and other products harmful to inhale and produced during the pre-heating processes.

The converter has been specially designed to operate together with the MESTRA HP furnaces, but it can also be connected to any other pre-heating furnace with an exhaust hose at its back of a maximum outer diameter of 38 mm.

Caution: before starting the unit, read these instructions.  
Store these instructions for further consultation.

## CAUTIONS

- Do not allow children or non-qualified personnel handling the unit.
- Do not disassemble repair or modify the unit.

- Do not touch the unit or the power cord with wet hands.
- Unplug the unit in case of storm and when you are not going to use it for a long time.
- Place the unit in a dry place, away from dust and water splashes.
- Do not pull, twist or fold the power cord excessively.
- Do not wet the unit; do not use chemical products to clean it.
- Disconnect the unit from the power source during the maintenance and cleaning operations.

- **WARNING:** to prevent electric shock hazards, connect the unit to a proper power source (grounded 230 V, 50/60 Hz).

- Do not touch the unit cover immediately after operating it. It may reach high temperatures.
- Be cautious with the converter to prevent burns even when it is not working.
- Make sure the power cord is not in contact with hot parts of the extractor or the furnace.

- Do not block the ventilation grille at the bottom of the unit or the outlet hole at the top.
- Do not place any object near the fan air intake, placed at the lower part of the catalytic converter. Leave at least 15 cm between the air intake and the nearest object.
- Remove any dirt or water remains from the cord connection.
- Unplug the unit immediately whenever you notice an unusual noise, smell or smoke.

- Avoid impacts or stresses that can cause mechanical problems.
- **WARNING:** do not turn the extractor fan upside down to prevent the catalytic converter cartridge getting out of its place and falling down.

- The metal edges of the cover may cause cuts. It is strongly recommended to wear gloves during the maintenance operation.

- The unit does not include a main switch. Bear in mind it can start up unexpectedly.

- Make sure the outlet hole of the converter is in a well ventilated place. Do not inhale these fumes directly. The most suitable place would be under an extractor hood with an exhaust hose to the outside.

- Contact the supplier for questions or detailed information about problems with the unit.

## INSTALLATION

1. Choose a well ventilated place in the lab to install the furnace. Remember that the catalytic converter treats the fumes produced in the furnace muffle, but it does not dispose of them. The most suitable place would be under an extractor hood with an exhaust hose to the outside. Depending on the type of fumes produced inside the furnace, it may emit pollutants even when using the catalytic converter. In this case, additional measures should be adopted to observe the local requirements.
2. The unit has been exclusively designed for dry operation. Do not install or store the converter in the open or in a humid place.

**WARNING:** do not turn the converter upside down to prevent the cartridge getting out of its place and falling down.

3. When you want to connect the converter to a MESTRA HP family furnace, insert the outlet hose placed at the back of the furnace to the intake hose of the converter. Then, tighten the fixing bolt in the intake hole of the converter.
4. If you want to connect the converter to a furnace not manufactured by MESTRA, make sure that the outer diameter of the outlet hose is not wider than 38 mm. Then proceed as in the previous section. If the diameters of the furnace outlet hose and the intake hole of the converter are very different, a better adjustment can be made by winding a piece of thermal insulator around the outlet hose of the furnace (maximum temperature 1200° C).

**WARNING:** never place the catalytic converter directly on the furnace. Make sure there is free space under the fan (15 cm) and over the unit (50 cm).

5. Connect the power cord of the unit into the power source. Make sure that the power cord plug is suitable for the country where the unit is to be used and the features of the power source match those in the identification plate of the unit.

**WARNING:** to prevent electric shock hazard, the unit must be connected to a suitable power source (grounded 230 V, 50/60 Hz).

6. The MESTRA HP furnaces feature a power outlet in their left side. If you connect the catalytic converter to this power outlet, it will operate automatically from 75° C up to the start of the second heating phase or until it exceeds 850° C. If you prefer a continuous operation of the converter, connect the unit to an independent power outlet.

Do not block the ventilation grille at the bottom of the unit or the outlet hole at its top.

## REPLACING THE CARTRIDGE

When the cartridge fails to work properly, replace it as follows when the cartridge fails to work properly, replace it as follows:

Do not touch the cartridge before making sure it has cooled down completely.

- Remove the cartridge and the insulator. Use pincers if necessary.
- Place the spare insulator inside the cartridge chamber, well fitted to the wall.
- Insert a spare cartridge.

**Make sure that the insulator is correctly placed around the cartridge.**

- Used cartridges contain toxic remains. Dispose of them in a suitable place.

**NOTE:** the cartridge of the converter is a perishable element and therefore it is not covered by the warranty.

## MAINTENANCE AND CLEANING

To ensure a long working life of the unit and an optimal operation, we strongly recommend:

- To clean the unit periodically with a soft cloth.
- To clean the wax and coal dust settled in the inner hose. To do it use a disposable cellulose cloth.

During the maintenance and cleaning operations keep the unit disconnected from the power source.

Do not touch the unit cover before making sure it has cooled down completely. It may reach very high temperatures.

## TECHNICAL SPECIFICATIONS

Height:	380 mm
Lower width:	185 mm
Upper width:	145 mm
Hose inner Ø:	38,5 mm
Weight:	5,3 kg
Voltage:	230V, 50/60 Hz
Power:	600 W
Fuse:	4 A

## DEFECT CHECKLIST

DEFECT	CAUSE	SOLUTION
<b>The fan does not work</b>	1. The furnace controls have failed to start the converter.	<ul style="list-style-type: none"> <li>▪ If you connect the converter to the power outlet of a MESTRA HP furnace, it will operate automatically from 75° C up to the start of the second heating phase or until it exceeds 850° C.</li> <li>▪ If you prefer a continuous operation of the converter, connect the unit to an independent power outlet.</li> </ul>
	2. Power cord unplugged.	<ul style="list-style-type: none"> <li>▪ Connect the power cord to a grounded 230 V, 50/60 Hz power source.</li> </ul>
	3. No current in the power source.	<ul style="list-style-type: none"> <li>▪ Make sure there is current in the power source.</li> </ul>
	4. The fuse is burnt out.	<ul style="list-style-type: none"> <li>▪ Replace the 4A fuse in the detachable fuse-box at the base of the connection plug.</li> </ul>
	5. Wire connections inside the unit.	<ul style="list-style-type: none"> <li>▪ Connect all the elements of the circuit properly.</li> </ul>
	6. The fan is damaged.	<ul style="list-style-type: none"> <li>▪ Replace the fan.</li> </ul>
<b>The resistor does not work</b>	1. Wire connections inside the unit.	<ul style="list-style-type: none"> <li>▪ Connect all the elements of the circuit properly.</li> </ul>
	2. The resistor is damaged..	<ul style="list-style-type: none"> <li>▪ Replace the resistor.</li> </ul>
<b>The converter emits an unusual big amount of fumes</b>	1. The cartridge is worn out.	<ul style="list-style-type: none"> <li>▪ Replace the cartridge of the converter.</li> </ul>

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