R-080021

E.....	ELECTROVIBRADOR “TEIDE 1”	7
GB	ELECTRO-VIBRATOR “TEIDE 1”	9
F.....	VIBRATEUR “TEIDE 1”	11
P.....	ELECTROVIBRADOR “TEIDE 1”	13

MESTRA®

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. (+34)944530388 - Fax (+34)944711725


E-mail: mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

Rev. 26/05/14

- | | |
|---|--|
| A | Bola de vaciado / Pouring ball / <i>Support-boule pour coulée</i> / Bola de vaziado |
| B | Plataforma de vibración / Vibration platform / <i>Plateau de vibration</i> / Plataforma de vibração |
| C | Regulador de potencia / Power regulador / <i>Potentiomètre</i> / Regulador potencia |
| D | Pata ventosa / Suction cap / <i>Pied-ventouse</i> / Pata ventosa |
| E | Interruptor general / Main switch / <i>Emplacement du support-boule</i> / Interruptor geral |
| F | Portafusibles / Fuse holder / <i>Porte-fusibles</i> / Porta fusíveis |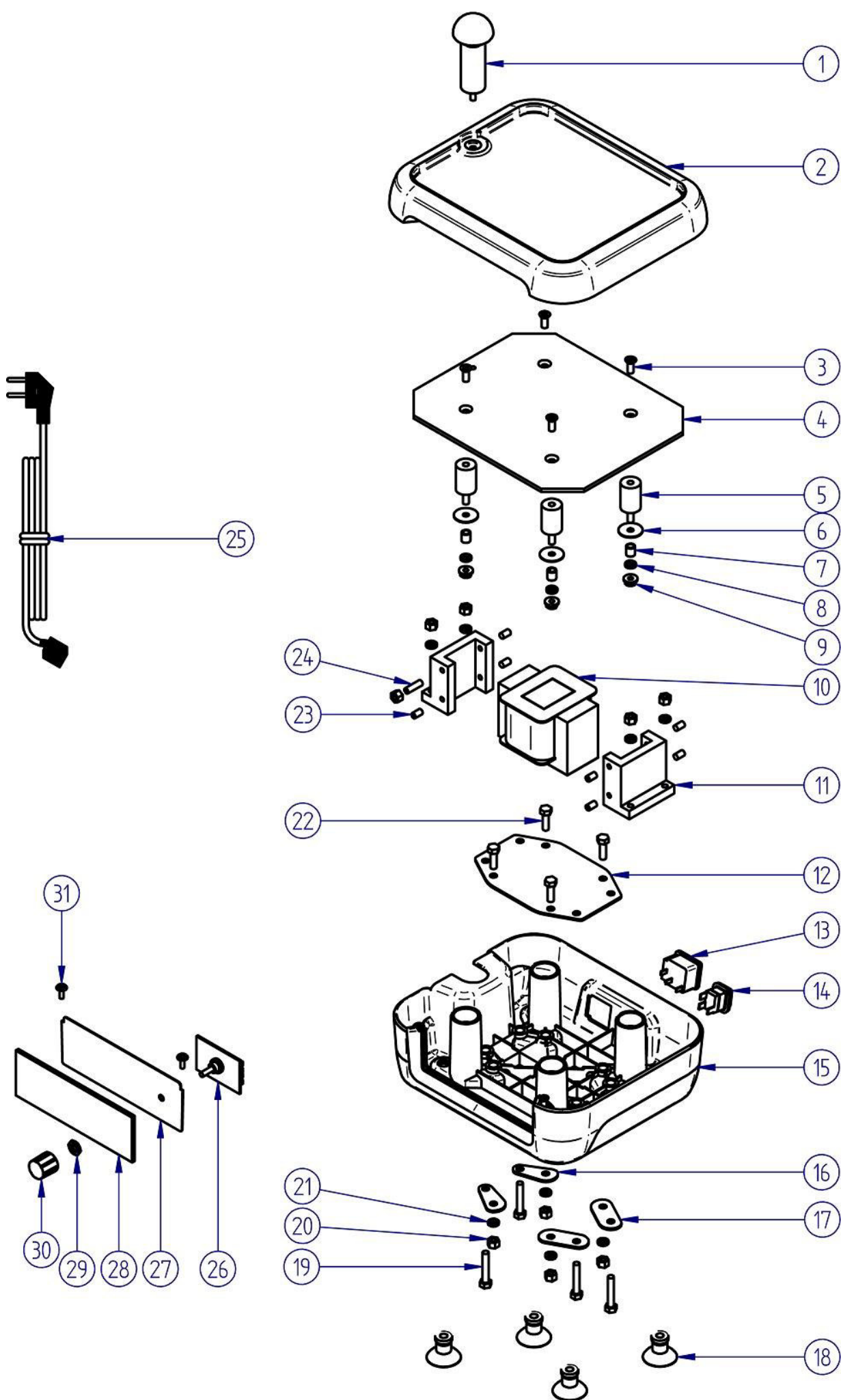

Nº	Ref.	DESCRIPCIÓN	DESCRIPTION
1	080022-10	Bola de vaciado	Emptying ball
2	080022-06	Plataforma de goma	Rubber platform
3	080020-34	Tornillo plataforma	Platform bolt
4	080022-07	Plataforma metálica	Metal platform
5	080020-31	Silent block	Silent block
6	080022-17	Arandela grande silent block	Silent block large washer
7	080022-01	Casquillo	Bushing
8	040064-62	Arandela pequeña silent block	Silent block small washer
9	080022-18	Tuerca con arandela	Nut with washer
10	080020-11	Electroimán 230 V	Electromagnet 230 V
	080020-11B	Electroimán 115 V	Electromagnet 115 V
11	080020-14	Soporte electroimán	Electromagnet support
12	080022-20	Soporte base	Base support
13	080490-06	Base de conexión	Connection base
14	080422-17	Interruptor general	Main switch
15	080022-03	Carcasa	Housing
16	080022-22	Soporte exterior pequeño	Small out support
17	080022-21	Soporte exterior grande	Large out support
18	080020-30	Ventosa	Suction cap
19	040064-32	Tornillo electroimán	Electromagnet bolt
20	080022-15	Tuerca	Nut
21	080022-14	Arandela	Washer
22	060064-28	Tornillo soporte	Support bolt
23	080020-15	Espárrago corto	Short stud
24	080010-16	Espárrago largo	Long stud
25	000250	Cable 230 V	Cable 230 V
	000252	Cable 115 V	Cable 115 V
26	080010-12	Tarjeta control 230 V	Control board 230 V
	080010-12B	Tarjeta control 115 V	Control board 115 V
27	080021-02	Soporte control	Control support
28	080021-01	Carátula control	Control cover
29	080010-18	Tuerca control	Control nut
30	080010-19	Pomo control	Control knob
31	080093-32	Tornillo control	Control bolt

E

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Ignacio Mestraitua
Director Gerente
Sondika, 01.03.2017

The R-080021, is a practical electro-vibrator unit specially designed for compacting materials for the jewellery and dental mechanics sectors.

NOTE: Carefully read this manual. It contains important safety information in the installation, use and maintenance. If the equipment is used in a manner not specified by the manufacturer, the security provided by the equipment may be compromised.

- Unscrew from the lateral housing and screw it in the upper hole in the vibration platform (B).


Meaning of the symbols on the unit

Danger. Risk of injury.

Read completely these instructions before operating the unit.

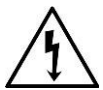Electric shock risk.

Risk of entrapment.

Installation

Environmental conditions:

- Use indoors only.
- Environmental temperature: 5-40 °C [41-104 °F]
- Max. relative humidity: 0-95 %
- Degrees of protection: IP20

- Place the unit on a smooth, firm, perfectly horizontal surface. Try not to install near to other devices, vibrations could be transmitted. If you do not have a surface like that, you can place the unit on a cloth or foamy material.
- Connect the plug to a 230 V, 50/60 Hz earthed power outlet.
- Place the device so that it is easy to use the disconnecting device (main switch).

Platform and pouring ball use


- Place the cylinder or items to be vibrated on the unit's vibration platform (B). In the case of impression bowls, the mixture can also be compacted by resting the bowl firmly on the edge of the unit and holding down the bowl handle with your hand.

- When the mixture is not fluid, the pouring ball can be useful. Lay the bowl on the ball to transmit vibration to the mixture.

Operation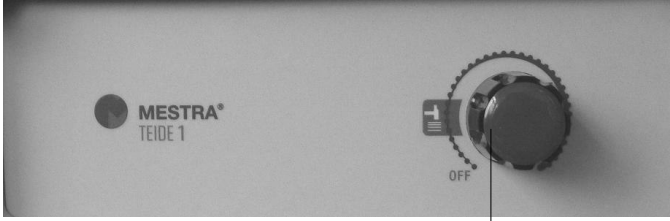

OFF
Vibration power

- Press the main switch placed in the back side.
- Slowly move the unit's regulator (C) clockwise and check that the oscillation rate gradually increases. Set the regulator to a suitable amplitude for the operation.
- After a short time, once the mixture is properly compacted, switch the unit off using the right switch.
- When not using the vibrator for a while, disconnect it from the network acting on the rear main switch.

Maintenance and cleaning

To ensure a long, trouble-free service life and performance of the unit, we recommend as follows:

- After using the unit, always clean away any splashes of mixture before they dry.
- As a precaution, always disconnect the unit from the mains before cleaning it.
- Regularly check the suction caps (D).
- Use only original spare parts supplied by the manufacturer.

Precautions

- Before connecting the unit to the power source, make sure the voltage is correct (grounded 230 V, 50/60 Hz).
- The cable may only be replaced by one with the same characteristics as supplied by the manufacturer. Do not use the unit in wet places or close to heat sources.
- Do not place easily broken containers or other objects on the vibration platform.
- Do not allow children or unskilled personnel to handle the unit.
- Do not pull, twist or bend the mains cable excessively.
- Before cleaning the unit, ensure that it is unplugged.
- Never immerse the unit in liquid when cleaning it.
- Do not place hot containers or objects on the vibration platform, as this may damage the rubber.
- Do not remove the vibration platform cover.

Technical characteristics

Power supply:	230 V, 50/60 Hz
Dimensions:	197 x 265 x 227 mm
Consumption:	240 W
Fuse:	Glass 9 A, 110-240 V
Sound power:	< 70 dBA
Weight:	5,5 kg
Vibrations p. m.:	3000

Declaration of conformity UE

The firm **Talleres Mestraitua S.L.**
with social address: Txori-erri Etorbidea, 60
48150 Sondika - Bilbao - Spain

Hereby declares that the product

Electro vibrator Teide 1, R-080164

Meets the requirements set out in the following directives:

- 2014/35/UE (low tension)
- 2014/30/UE (electro magnetic compatibility)
- 2006/42/CE (machinery safety)

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