
R-100360

ES	4	Manual original
EN	6	
FR	8	
DE	10	
PO	12	
IT	14	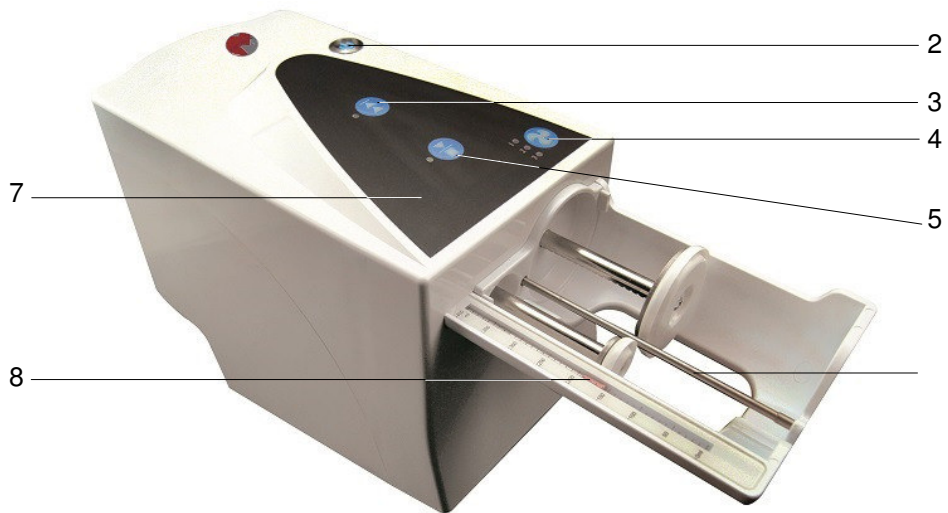

1. Toma de conexión con fusible / **Power socket with fuse** / *Prise avec fusible* / Anschluss mit Sicherung / **Tomada de ligação com fusível** / *Presa con fusibile*
2. Interruptor general / **Main switch** / *Interrupteur général* / Hauptschalter / **Interruptor geral** / *Interruttore generale*
3. Retroceso y parada / **Back and stop** / *Retour -arrêt* / Rücklauf und Halt / **Retrocesso e paragem** / *Ritorno e stop*
4. Selección de velocidad / **Speed selection** / *Sélection de la vitesse* / Geschwindigkeitsanwahl / **Seleção da velocidade** / *Selettore velocità*
5. Avance y parada / **Forward and stop** / *Progression-arrêt* / Vorlauf und Halt / **Avanço e paragem** / *Avanti e stop*
6. Husillo / **Drive rod** / *Injecteur* / Spindel / **Fuso** / *Alberino*
7. Teclado / **Keyboard** / *Panneau de commandes* / Tastatur / **Teclado** / *Tastiera*
8. Visor de capacidad / **Capacity display** / *Viseur de capacité* / Füllstandsanzeige / **Visor de capacidade** / *Visore capacità*

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NOTE: Carefully read this manual. It contains important safety information in the installation, use and maintenance. If the equipment is used in a manner not specified by the manufacturer, the security provided by the equipment may be compromised.

Meaning of the symbols on the unit

Read completely these instructions before operating the unit.

Intended use

The R-100360 is a mixer specially designed for use with dental impression silicones.

Do not use the machine for a different purpose.

Precautions

- Keep this manual in a suitable place to consult it in the future.
- Check the overall state of the unit just upon unpacking it.
- The damage caused by a faulty installation or usage of the unit will not be covered by the warranty.
- the risk of electric shock, this equipment must only be connected to a properly grounded power source. Before connecting the appliance, check that the identification label indicates the correct voltage. The installation must comply with all applicable safety standards.
- Remove any dirt or water remains from the cord connection.
- Do not pull, twist or fold the power cord excessively.
- Do not touch the unit or the power cord with wet hands.
- Place the unit in a dry place, away from dust, water splashes and flammable materials (alcohol, solvents, etc.).
- Unplug the unit immediately whenever you notice an unusual noise, smell or smoke.
- Do not allow children or non-qualified personnel handling the unit.
- Do not use the unit if any doubts. Contact the supplier for questions or detailed information about problems with the unit.
- Do not dismount, repair or modify the unit.
- Unplug the unit in case of storm and when you are not going to use it for a long time.
- Unplug the unit before cleaning, do not use chemical products.

- Contact the supplier for questions or detailed information about problems with the unit.
- Before connecting the device, make sure it is placed on a firm and solid surface.
- The cord may only be replaced by one with the same characteristics as supplied by the manufacturer.
- Wear proper working cloth. Loose cloth, hanging or long hair could be caught by the machine.
- If the motor shaft locks during normal operation, disconnect the unit from the mains before attempting to release it by turning it manually.
- Do not use the apparatus near solvents or other flammable materials.

Installation

Environmental conditions:

- Use indoors only.
- Environmental temperature: 5-40 °C [41-104 °F]
- recommended temperature: 18-40 °C [64-104 °F]
- Max. relative humidity: 0-80 %


1. Carefully unpack the model trimmer. Make sure the unit has not suffered any damage during the transport.
2. Place the unit on a flat and firm surface, away from any heat or vibration source.
3. The unit has four rubber pads. When designing the unit, special care has been taken to give the unit enough weight and robustness to prevent any movement during its operation. No additional fixing element is needed.
4. Connect the unit to a proper and grounded power source.

Operation of the unit


1. Connect the power supply port.


2. Remove the cap or the cartridge.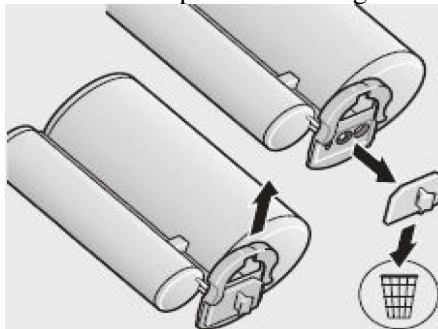


- Fix the mixing tip on the cartridge and make sure it is perfectly secured in the cartridge.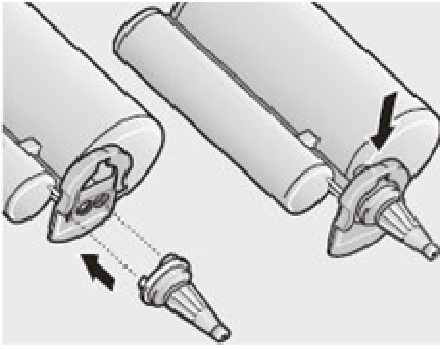


- Put the unit switch on (2).
- Put the cartridge into the machine.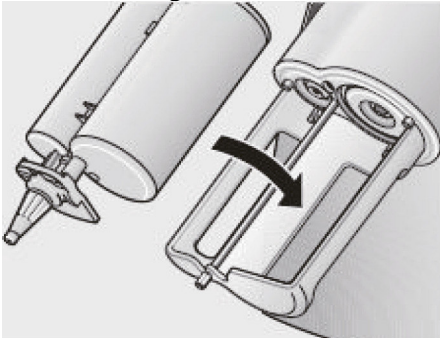


- Confirm the same line between the centres of cartridge and plunger disks.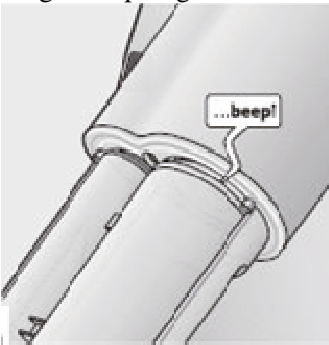


- Finish installation and start work.

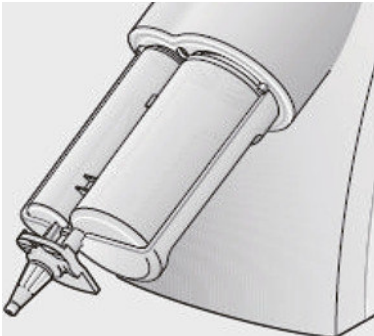- To start and stop material injection, press button (5).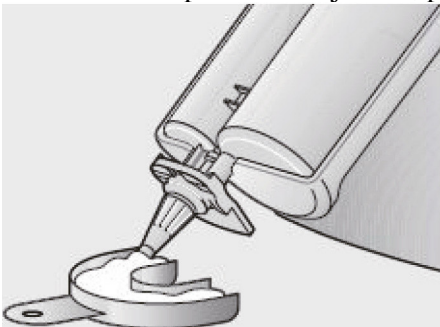

- Finish injection and press back button (3) to release the cartridge before removing.

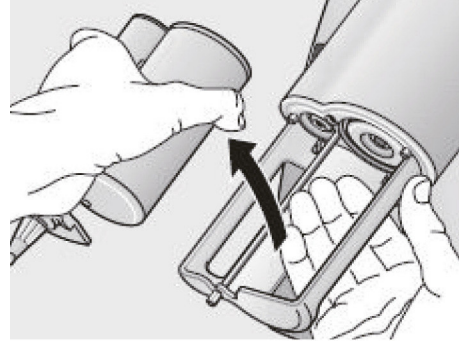- Work finished, put the unit switch off (2).
- Después del uso, se recomienda realizar una limpieza para eliminar los restos de material.
- After operating the mixer, it is strongly recommended to clean the remains of material.

Maintenance and cleaning

To ensure a correct, reliable and durable performance of the unit we strongly recommend:

- Use a new nozzle for each process.
- Make sure that the nozzle is perfectly secured in the cartridge.
- Make sure the cartridge is perfectly aligned with the pushers.
- If the cartridge was stored for a long time, make sure that the material outlet has not been blocked.
- After use, clean any remaining dust and debris from the machine.
- Do not use alcohol or other liquid solvents to clean the appliance.
- Never immerse the appliance in any liquid.
- If you notice abnormal noises or vibrations, send the device to a service center.
- The unit has been designed to operate correctly leaning on the four rubber blocks in its base.

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