Ref. 010160 / Ref. 010170

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The easy locking articulators (Ref. 010160 for full dentures and Ref. 010170 for partial dentures) are two new innovative articulators that save working time.

PROCESSING INSTRUCTION

1. Trim a rough plaster cast:

- In case of Arti-S, trim equally the width of the upper and lower plaster cast as shown in Fig. 1.
- Also trim equally the end part of upper and lower plaster cast as shown in Fig.2 (in case of full mouth model).
- Use part No. 205, 206 rubbers to make plaster cast easily.

Fig. 1

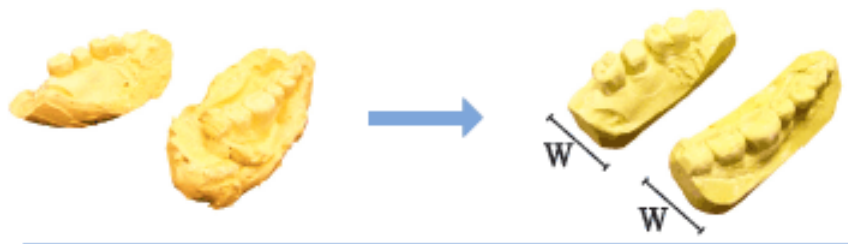
Fig. 2

2. Fix upper & lower plaster cast with using clamping screws. (Fig3)

Fig. 3

3. Connect and fix the upper and lower plaster cast on the right position with using dental wax. (Fig4)

4. Please hold the ball holder carefully and screw on the ball lightly bolt with using screw driver.

Do not contact the end of ball holder to the neck of the ball as shown Fig. 5.

CAUTION

- ◆ Do not drop and not force excessively the articulator in order to prevent losing bolts.
- ◆ Do not force excessively when fix plaster cast by screw.
- ◆ Do not use too big, small and weak plaster cast.
- ◆ Keep the articulator free from moisture to protect screws of articulator from corrosion.
- ◆ Use articulator parts in accessories and can order them separately if need more.

MESTRA®

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA