
Ref. 150130

ES.....	3	Manual original
EN.....	5	

- |   |   |
|---|---|
| A | Rodillos / <b>Rollers</b>                   |
| B | Control de velocidad / <b>Speed control</b> |
| C | Interruptor general / <b>Main switch</b>    |
| D | Luz encendido / <b>Power light</b>          |

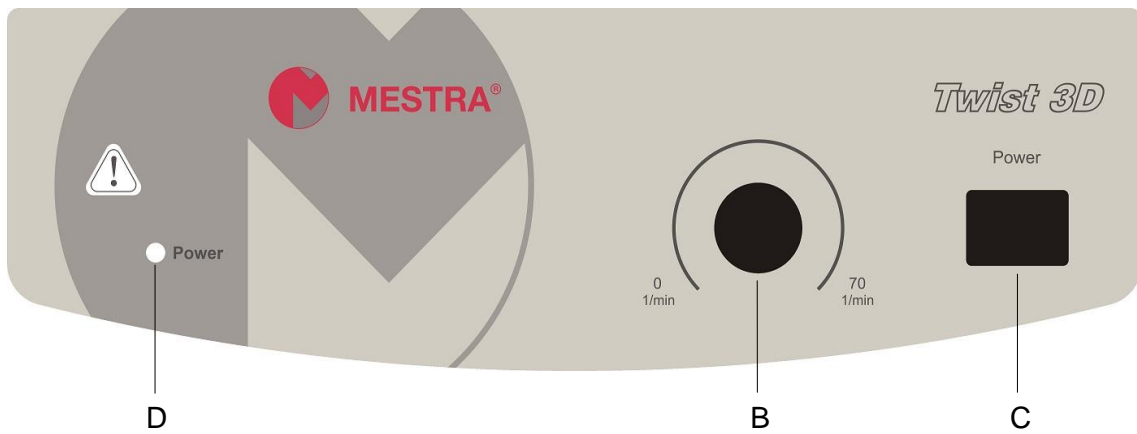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## Warnings

### Preventive measures

- Attention: before using the machine, please completely read this operating manual.
- Keep this operating manual properly for future.
- Do not allow children or non-qualified personnel handling the unit.

- Do not touch the unit or the power cord with wet hands.
- Unplug the unit in case of storm and when you are not going to use it for a long time.
- Place the unit in a dry place, away from dust and water splashes.
- Do not pull, twist or fold the power cord excessively.
- Do not wet the unit; do not use chemical products to clean it.

- **WARNING:** to prevent electric shock hazards, connect the unit to a proper power source (grounded 230 V, 50/60 Hz).

- Wear personal safety guards to avoid the risk of splashing liquids.

- Do not touch the rollers when they are working.

- The machine features a precision circuit. Prevent fortuitous impacts and stresses that could cause mechanical damage.

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- Do not use the machine with products different than defined in the intended use.
- To prevent accidents, be cautious when using the unit.
- Always keep the device in idle mode, except when mixing.
- Choose a flat and firm surface to place the unit, away from heat or vibration sources.
- Do not dismount, repair or modify the machine.
- If you find a defective component or a problem do not use the machine.

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## Intended use

The ref. 150130 is designed for mixing viscous substances at dental labs and clinics.

For optimum output and performance, we advise you to read the following instructions with great care.

## Installation

- Carefully unpack the unit. You will find the following fittings inside the chamber:
  - Main unit.
  - Power cord.
  - User manual.
- Ambiental conditions:
  - Temperature: 5-40 °C.
  - Relative humidity: 80 %.
  - Protection class: IP 21
- Connect the electric plug to 230 V, 50/60 Hz, earthed.

## Operation

1. Switch the unit ON (C).
2. Rotate the seed control knob to set the speed.

## Maintenance and cleaning

To ensure a lengthy service life, we recommend the following:

- Unplug the unit before cleaning.
- Do not spray cleanser into the instrument when cleaning.
- Use only recommended cleansers (water containing tenside, isopropyl alcohol).
- If you require maintenance service, the unit must be cleaned in advance to avoid pollution of hazardous substances. Send it back into original packing.
- If the unit will not be used for a long time, switch it off and place in a dry, clean room temperature and stable location).

## Technical characteristics

Power supply: .....	100-240 V, 50/60 Hz
Power: .....	25 W
Height: .....	103 mm
Width: .....	253 mm
Length: .....	480 mm
Weight: .....	3,12 kg
Number of rollers: .....	6
Roller dimensions: .....	Ø30 x 280 mm
Roller Turn Amplitude: .....	24 mm
Max. load: .....	4 kg

**MESTRA<sup>®</sup>**

*Your trusted brand*

## Talleres Mestraitua S.L.

Txori-erri Etorbidea, 60  
T: (+34)944530388 - F: (+34)944711725  
mestra@mestra.es - www.mestra.es  
48150 SONDIKA - BILBAO - ESPAÑA