Ref. 100192

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Ref. 100192 is a practical static steam generator, with a tank capacity of 2 liters and an operating pressure of 4 bar. It has been designed and built for use by dental technicians and jewelers. The tank is entirely made of stainless steel.

INSTALACIÓN

- Carefully, unpack the machine. You will find a filling glass and a practical funnel.
- Visually inspect, paying particular attention to possible damage of the cables and steam nozzle.
- Choose for the location of the appliance a horizontal, flat and rigid base, away from sources of heat or vibrations, capable of supporting a load of 9 to 10 kg and larger than the dimensions of the device base.
- Make sure that the ventilation grills are free. Keep a clearance of more than 15 cm from the wall or any other object.
- Screw the vaporizing nozzle into the corresponding hole on the front panel.
- Connect the foot pedal to the back of the machine.
- Before connecting the machine, check that the nameplate indicates the correct voltage for the power supply.
- Connect the appliance to a grounded 220-230 V, 50/60 Hz socket.

FILLING THE BOILER

1. Turn off the front panel switch.
2. Make sure the device is cool and is not plugged into a power socket.
3. Turning counterclockwise, and open the water tank cap.
4. Using the funnel and measuring cup provided in the box, fill the tank. Be careful to avoid overfill the water tank (maximum capacity 1 L of water).

Warning: Do not use water that contains perfume, chemical solvents or additives to remove limescale, as they can damage the appliance. **Use only distilled water.**

Recomendation: hot water can be used if available, for refilling. In this case, the apparatus will be ready to operate in a shorter period of time.

5. Close the water tank cap, being careful not to over-tighten it.

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6. Make sure the switch on the front panel of the device is off. Connect the device to a suitable electrical socket.

USE OF THE APPLIANCE

1. Turn the switch located on the front panel of the device.
2. The red light will light, indicating that the boiler is heating up. Wait until the green (steam ready), lamp on the front panel lights up. The time required depends on the initial temperature of the water and varies between 5 - 10 minutes. Once the green lamp lights up, your device is ready to be used.

Warning: Do not open the cap of the water tank during the operation of the machine. Wait until the machine is completely cool.

Note: It is normal for the lamps green and red, lights and turn on/ off during operation indicating heating periods.

3. Once the green lamp lights up, push the operation pedal. The steam will start to flow. Bring the object closer to the steaming nozzle at a distance of about 10 - 20 cm. and move it to impregnate it with the steam.

Note: The steam coming out of the steam gun may contain water particles at the first few seconds due to condensation on the nozzle.

END OF OPERATION

- After you have finished using the machine, turn off the switch located on the front panel.
- Unplug the power cord from the socket and wait until the device cool down.
- Once the device has cooled down, empty any remaining water into the tank. Small amounts of water or moisture that may remain in the water tank will not damage the device.
- If the device is not going to be used for a long time, boil the water in the tank to evaporate it completely without closing the cap. Let it cool down and, wrapping the power cord, and store the device in a suitable place.

SAFETY DEVICES

The safety devices in this machine are: cap with safety valve, pressure switch, thermostat on the heating element.

1. In the absence of water, a thermostat stops the heating of the resistance.
2. A pressure switch prevents overpressure in the boiler.
3. A thermostat prevents overheating.
4. A safety fuse is activated in case of a malfunction of the bimetallic thermostat. In such cases, the safety fuse cuts the electrical power to the boiler. If it happens, contact an authorized technical service.
5. In the event of the above systems fail, a safety valve automatically releases the steam into the boiler.

PERIODIC CARE

In normal use, depending on the hardness of the water, there may be calcification and sedimentation in about a month. Therefore, apply periodic maintenance once a week or after every ten uses:

- Using a cleaning cloth, wipe off any oil or dirt deposits that may be on the exterior surfaces.
- Visually inspect steam nozzle, cap seal, foot pedal, switch. If you detect any visible damage or malfunction, contact your authorized dealer immediately.
- Fill the water tank halfway, rinse with water and empty. Repeat this step two more times.

Attention: You can use tap water to clean the appliance. Never use solvents or chemicals that could damage the boiler tank or the appliance body.

MAINTENANCE AND CLEANING

To ensure optimum reliability and a long life for the appliance, we recommend the following tips:

- It is necessary to periodically decalcify the appliance:
 - Open the cap.
 - Fill with clean water the tank, shake and empty.
 - Repeat the operation as many times as necessary.
 - Refill the boiler with distilled water.
- It is advisable to wash the appliance tank once a month, proceeding as follows:
 - Disconnect the appliance from the mains.
 - Open the cap of the appliance slowly so as to allow any residual steam to escape.
 - Put ½ liter of water in the tank using the funnel supplied.
 - Let the water act for a few minutes.
 - Empty the water by overturning the container.
 - Repeat the operation if necessary.
- Some calcium deposits may stick to the steam nozzle. With the tank cap open, clean it by blowing compressed air to release the nozzle tube.
- In the event of a breakdown or breakage of the machine, do not intervene: contact your **MESTRA** dealer.

PRECAUTIONS

- Before connecting the appliance, make sure that it is a 220-230 V, 50/60 Hz socket with earth connection. It is also necessary that socket is well protected with a thermal switch. Consult your installer.
- Do not allow children or unqualified people to handle or come near the appliance when it is operating.
- Place the appliance on a rigid and flat surface with no risk of tipping over.

- Make sure the vents are clear. The nearest wall or object must be at least 15 cm away.
- Disconnect the appliance from the socket before cleaning.
- Unplug the device from the electrical socket when not in use.
- Do not operate the device if the power cord or steam hose is damaged. Call your authorized dealer for replacement of any damaged parts.
- The upper part of the appliance can become very hot during use.
- The temperatures of the metal part of the steam nozzle will be quite high, avoid any contact.
- The device has the ability to generate continuous steam at high temperature and large volumes. Avoid close contact with the steam and do not point the steam directly at the face or hands.
- Point the steam jet only in the direction of the job you are performing.
- Handle the machine with great care. It can be damaged due to a falling impact.
- Do not immerse the device in water or other liquids.
- Do not fill the boiler with any substance other than distilled water.
- Do not exceed the maximum water level.
- Before filling or emptying the water tank, unplug the device from the socket and turn off the boiler switch.
- For your safety, do not open the cap before the pressure in the device drops to zero. For refilling, follow the above mentioned instructions.
- Open the cap of the appliance with caution. The sudden release of the water steam accumulated in the tank could cause burns.
- Do not overtighten the tank cap.
- If the cap is damaged, contact an authorized dealer and replace it.
- Do not try to repair the appliance by yourself. Do not disassemble it. For repair, take your machine only to authorized dealers.

TECHNICAL CHARACTERISTICS

Height:	300 mm
Height (con asa):	350 mm
Width:	180 mm
Depth:	285 mm
Depth (with tap):	380 mm
weight (with pedal):	6100 g
Power supply:	220-230 V, 50/60 Hz
Power:	1300 W
Pressure:	4 bar
Temperature:	135 °C
Tank capacity:	2 L

MESTRA[®]

Your trusted brand

Talleres Mestraitua S.L.

Txori-erri Etorbidea, 60
T: (+34)944530388 - F: (+34)944711725
mestra@mestra.es - www.mestra.es
48150 SONDIKA - BILBAO - ESPAÑA